







Navigating SmartFocus™

SmartFocus™ helps you enjoy precise, personalized hearing with unprecedented control of your listening preferences.

	Clarity	Comfort
What are you trying to achieve?	<ul style="list-style-type: none"> • Increase clarity of sounds, such as speech • Focus on sounds in front of you 	<ul style="list-style-type: none"> • Increase listening comfort • Decrease annoyance and loudness of sounds
How do you adjust the control?	<p>⬆️ Scroll the dial on the Smart Control or hearing instrument upwards, one click at a time, until desired listening result is achieved</p> <p><i>Note: When you reach the maximum clarity setting, you will hear two beeps</i></p> 	<p>⬇️ Scroll the dial on the Smart Control or hearing instrument downwards, one click at a time, until desired listening result is achieved</p> <p><i>Note: When you reach the maximum comfort setting, you will hear two beeps</i></p> 
What is happening to the hearing instrument?	<ul style="list-style-type: none"> • Spoken words are enhanced • Your hearing instruments automatically adjust to focus on / emphasize sounds in front of you • Awareness of sounds beside and behind you is reduced 	<ul style="list-style-type: none"> • All sounds are lowered to improve listening comfort. Sounds detected as noise are lowered further • Your hearing instruments automatically adjust to provide awareness of sounds all around you, while ensuring listening is comfortable 
When might you need it?	<ul style="list-style-type: none"> • Attending a lecture • Watching television • Engaging in conversation with a companion at a busy restaurant 	<ul style="list-style-type: none"> • Walking along a busy street • Riding the train • Waiting for the show to start in a crowded theater

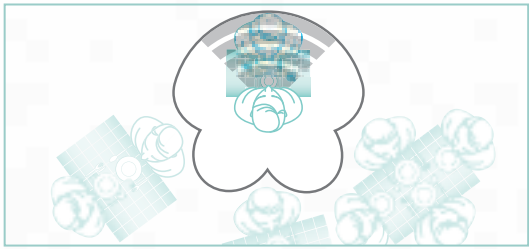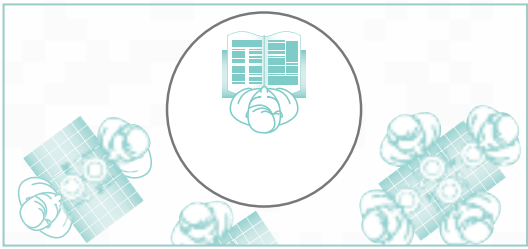

Taking control of your hearing success

Smart Control is a remote control that enables you to simply and discreetly adjust the various functions on your hearing instruments.

1 On/Off switch (travel lock)

To avoid accidental activation of your Smart Control, slide the switch to 'off' (locked) when it is in your handbag or pockets.

2 Telephone button (or program button)

Switches your hearing instruments to the telephone program, or 'other' program assigned by your hearing healthcare professional.

3 LearnNow™

Any time you are happy with a volume control or smartFocus adjustment you make in a particular environment, press the button to instantly teach your hearing instrument your listening preferences. The next time you encounter a similar listening situation, your hearing instruments will automatically adjust to your preferred settings.*

4 Left-both-right slider

This switch allows you to select the hearing instruments you want to adjust: left, right or both.

5 SmartFocus™

Scroll the wheel upwards to increase **clarity**, or downwards to increase **comfort**, in a particular listening situation.

6 Volume control

If the overall sound level is too soft, press + to increase loudness.

If the overall sound level is too loud, press – to decrease loudness.

7 Home

Quickly returns your hearing instruments to the volume and smartFocus settings from the beginning of your day.

8 Program change button

Change listening programs with the simple push of a button.

Important tips for getting the most out of SmartFocus™

Break through the noise – The adjustments you make to smartFocus will be most noticeable in noisy environments.

Take your time – Scroll the smartFocus wheel slowly, one click at a time. After each click, wait about 4-5 seconds to let the hearing instrument adapt to the new settings. Several clicks may be required to achieve your desired result.

Avoid getting cornered – Try to avoid sitting with your back to a wall in any listening situation. SmartFocus will be most beneficial if the sounds are coming from all around you. Try to position yourself so the person you want to hear is in front of you and undesirable sounds, such as noise, are behind you.

There's no place like home – Press the 'Home' button on your Smart Control when you want your hearing instruments to return to the volume and smartFocus settings from the beginning of your day.

* LearnNow works in the automatic program only.